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1. Intended use

The optical fibers (laser fibers) are designed for use in connection with the Calculase and Calculase II laser units from KARL STORZ in the wavelength spectrum of 2100 nm.

They are **intended for use** in operating theaters and doctors' offices for human lithotripsy applications and for the ablation and vaporization of tissue.

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No **contraindications** directly related to the product are currently known.

The laser fiber must not be used if, in the opinion of the responsible physician, the health of the patient is endangered through such use, e.g., due to the general condition of the patient.

Improper use (other than the Intended Use) is prohibited for safety reasons.

Unauthorized conversions or modifications to the laser fiber are not permitted for safety reasons.

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WARNING: *Risk of infection and injury: This laser fiber is sterile when delivered and can be resterilized up to 9 times. Due to its special design and material properties, it may not be reprepared or reused after it has been used for a tenth time.*

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Modelle/Models/Modelos 27750271, 27750272, 27750273, 27750287

2. Symbols employed

WARNING: A Warning indicates that the personal safety of the patient or physician may be endangered. Failure to observe a Warning could result in injury to the patient or physician.

CAUTION: A Caution indicates that particular service procedures or safety precautions must be followed to avoid possible damage to the laser fiber.

NOTE: A Note indicates special information about the operation of the laser fiber, or clarifies important information.

3. Safety instructions

WARNING: Risk of infection and injury: Only use the laser fiber if the packaging is intact and the expiry date has not been exceeded.

CAUTION: Risk of material damage: Use the STORZ fiber trimming set when handling laser fibers for intraoperative preparation of the distal end of the laser fiber.

CAUTION: Risk of material damage when inserting the laser fiber into endoscopes. Only insert the laser fiber when the working channels are straight. No laser light emission inside the endoscope.

WARNING: Risk of injury to the skin and eyes from laser light. Safety glasses are to be worn in accordance with the information in the laser unit's instruction manual.

WARNING: Risk of fire: Laser light from Class 4 lasers represents a source of ignition. When using the laser fiber take suitable measures to prevent laser-induced fires and explosions.

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**Calculase II laser fiber
(laser fibers for multiple use)**

Modelle/Models/Modelos 27750271, 27750272, 27750273, 27750287

! **WARNING:** Risk of injury if the maximum output and energy of the laser fiber is exceeded. Do not exceed the maximum output and energy (see table below) for the laser fiber. Only use laser fibers with the stated laser units.

Art. no	E _{max} [J]	P _{max} [W]
27750271	1,2	12
27750272	2	20
27750273	2	20

4. Application
➤ Remove the laser fiber from the packaging.

- Open the sterile packaging.
- Under aseptic conditions carefully remove the laser fiber together with the laser fiber carrier from the sterile packaging.
- Remove the distal laser fiber end from the carrier and carefully unwind the laser fiber from the carrier.
- Remove the connector from the holder on the carrier.
- Take the cap off the SMA connector.

! **CAUTION:** Risk of material damage: Do not touch the light entrance surface.

! **CAUTION:** Risk of material damage: Do not damage the proximal laser fiber end.

➤ Plug the connector into the laser unit

- To do so, plug the connector axially into the fiber socket on the laser unit.
- Fix the connector with the nut and tighten by hand.

! **CAUTION:** Risk of material damage: Do not tilt the laser fiber connector. Do not use tools.

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**Calculase II laser fiber
 (laser fibers for multiple use)**

Modelle/Models/Modelos 27750271, 27750272, 27750273, 27750287

5. Preparation

1. Attach the protective cap to the SMA connector.

» Cleaning

1. Use a low lint disposable cloth to remove all heavy soiling from the laser fiber.
2. Remove any tissue remnants adhering to the laser fiber by cutting the laser fiber at these points.
3. Perform wipe-down disinfection of the laser fiber with an alcohol-based rapid action disinfectant or an ammonium chloride-based cleaning agent.

» Trimming

1. Prior to sterilization, trim cleaned laser fibers at the distal end.
2. To do so, cut approx. 5 cm off the laser fiber.
3. Remove 3 - 4 cm from the outer fiber jacket (buffer) using a suitable fiber stripper.
4. Trim the optical fiber using a ceramic knife and a silicone mat in accordance with the instructions for the fiber trimming set.
5. Dispose of laser fibers which show signs of damage at the SMA connector (to do so, remove the protective cap).
6. To conclude, always check the quality of fiber trimming using an aiming beam.

» Sterilization

 **NOTE:** The laser fiber can be sterilized when the counter is higher than zero.

1. Cut one number off the sterilization counter prior to each sterilization process.
2. If necessary, attach the protective cap to the SMA connector.
3. Wind the laser fibers up with a radius of ≥ 19 cm.
4. Package the laser fibers into suitable packaging.
5. Sterilize the laser fibers in autoclaves at 134°C (4 min – 5 ½ min).

 **CAUTION:** Risk of material damage: Sterilization may be performed a maximum of 9 times. Do not sterilize the laser fibers more than a total of 9 times.

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Calculase II laser fiber (laser fibers for multiple use)

Modelle/Models/Modelos 27750271, 27750272, 27750273, 27750287

6. Test for proper functioning

After having properly connected the laser fiber connector to the laser unit, check optical transmission of the laser fibers.

1. Switch the laser unit on.
 2. Connect the laser fiber.
- ☛ The aiming beam is now available.

Calculase red,

Calculase II green.

3. Check the distal reproduction of the aiming beam light.

☛ Optical laser fiber transmission is **OK**

Optical laser fiber transmission is **OK** if the aiming beam produces a round, evenly bright mark against a white or light backdrop at a distance of 5 to 10 cm from the distal fiber end.

☛ Optical laser fiber transmission is **not OK**

Optical laser fiber transmission is **not OK** if the mark produced is not round, is unevenly bright or is extremely irregular (comet tail).

1. In this case, check the proximal laser fiber surface and correct trimming.
- ☛ If you are still unable to achieve a round aiming beam, the laser fiber must be replaced if treatment results prove inadequate.

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7. Description

Depending on the fiber type, various pulse energies and medium laser outputs can be applied (see instruction manual of the Calculase and Calculase II laser units).

The fibers are quartz fibers with a protective plastic exterior. They can be trimmed intraoperatively.

1. Remove the protective jacket using the respective fiber stripper from the fiber trimming set.
2. Trim the tip of the fiber using the ceramic knife (see Calculase instruction manual).

8. Removing the laser fiber

1. Switch the unit to standby mode.
2. Remove the laser fiber.

WARNING: Risk of infection and injury: The laser fiber is sterile on delivery and can be resterilized up to nine times.

3. Prepare the laser fiber if it is not yet ready for use (can be prepared 9 times).

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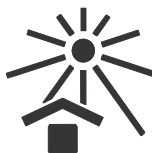
10. Handling

- Note the expiry date.
- Only open the original packaging immediately before using the product.
- On removing the product from the packaging check for any signs of damage: Damaged packaging may compromise the sterility of the product.
- When removing the product from the primary packaging, the rules of aseptic handling must be followed.
- The product must be examined for damage, such as cracks, deformations, etc. Defective products must never be used.
- During handling, the product must not come into contact with objects or chemicals which could damage it or change its surface.

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11. Storage conditions

The product must be stored in a clean and dry place away from direct sunlight and must not be used after the expiry date.

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12. Directive compliance

This medical product bears the CE mark in accordance with the Medical Device Directive (MDD) 93/42/EEC. A code number after the CE mark indicates the responsible office.

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